

IPR Set | 4594



For interproximal enamel reduction (IPR) according to Dr. Drechsler.


These days, interproximal enamel reduction (IPR) is increasingly used in all sectors of modern orthodontics. This not only applies to all fixed buccal and lingual appliances but also to treatments with aligners. Interproximal enamel reduction has become an invaluable aid in orthodontics, preventing extractions and ensuring successful straightening of teeth, especially in adults.

A new, professional IPR Set has been developed in close collaboration with Dr. Thomas Drechsler. This kit combines easier handling with an optimised clinical treatment, considerably facilitating the work of orthodontists and dental assistants.

The patented oscillating discs come clearly arranged in a modular storage box with a practice-orientated design. Suitable for sterilisation, this ergonomic storage box meets all criteria of efficiency and hygiene in dental practices.

Thanks to the different disc sizes and their logical arrangement in the storage box, it is now guaranteed that the amounts of inter-proximal substance that usually need to be removed (i.e. 0.2, 0.3, 0.4 or 0.5 mm) can be defined and achieved even more accurately.

A range of oscillating discs – coated on one or both sides – are available to remove any amount of interproximal substance between 0.2 and 0.4 mm as required. This easy step-by-step technique not only produces reliable, reproducible results, it also provides greater comfort to both orthodontist and patient during enamel reduction.

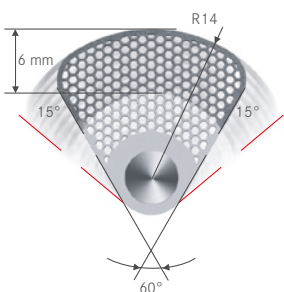

German patent DE 197 54 879

Scientific advice:

Dr. Thomas Drechsler,
 Fachpraxis für Kieferorthopädie
 Orthodontic Clinic · Wilhelmstraße
 40 65183 Wiesbaden
www.kfo-wiesbaden.de

Application:

1. Gentle, manual separation of the contact point: first of all, a thin abrasive strip WS37EF with honeycomb design coated with extra fine grit (yellow) is inserted between the two teeth.




2. This is followed by a slightly thicker strip WS37 (0.13 mm) coated with medium grit (blue).




3. Once the interproximal contact has been eliminated with the abrasive strips, the first oscillating disc OS1FV (0.13 mm, coated on one side only) can be used without jamming. The oscillating angle of the discs is 30° (15° in each direction). The sequence of use is such that each disc is 0.05 mm thicker than the previous disc. This allows a smooth, gradual increase of the interdental distance in complete safety.




4. This is followed by the oscillating disc OS15FV, which is – like its predecessor – only coated on one side. This allows the selective removal of dental enamel on one side of the tooth only, according to requirement.

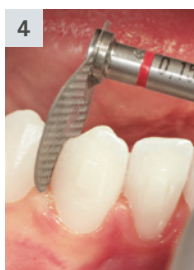


Oscillating discs with coating on their rear side only are available for work on distal surfaces (figure numbers feature the letter “H” instead of “V”). If an even substance removal on both sides is required, the oscillating disc OS15 with coating on both sides (thickness: 1.5 mm) is recommended.

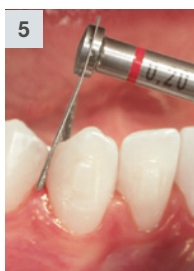

If you intend to reduce the dental enamel by a total of 0.2 mm, just use the discs in the first section of the storage box marked 0.2. Quando la riduzione prossimale dello smalto prevista è di 0,2 mm in totale, sono sufficienti i dischi posizionati nel primo settore del box contrassegnato con 0.2.

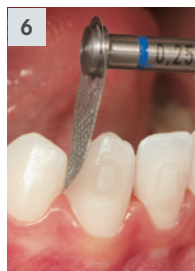


To enable the removal of the exact amount of enamel, the thickness of the oscillating discs is 0.05 mm less than the required dimension. This is because the enamel reduction is followed by an interproximal polishing step. Together, these two steps will result in the removal of the exact amount of enamel required, eliminating the need for a subsequent dimensional check.

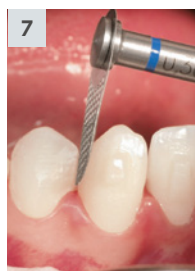

If you wish to reduce the dental enamel by 0.3 mm or 0.4 mm, use the discs in section 0.2 first and work through the subsequent sections – starting at the top of each sector – until you have reached the last disc at the bottom of the section that corresponds to the required target size:

- 5. Oscillating disc OS20F coated on both sides (thickness: 0.20 mm),
- 6. ... oscillating disc OS25M coated on both sides (thickness: 0.25 mm),

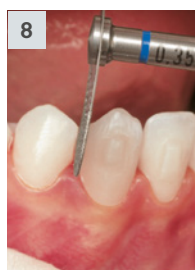


- 7. ... oscillating disc OS1M coated on both sides (thickness: 0.30 mm), and

- 8. ... finally, the oscillating disc OS35M coated on both sides (thickness: 0.35 mm) is used.





9. Result after use of the oscillating discs.

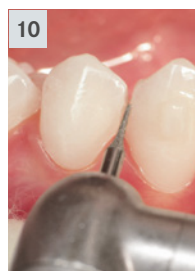

10. The IPR Set also contains a rotary diamond instrument 8392.314.016 (red ring) if the interproximal enamel is to be reduced by more than 0.5 mm. This instrument is stored in the section marked 0.5. Applied horizontally, this finisher has a particularly short working part to enable quick, yet precise interproximal enamel reduction between neighbouring teeth.

11. Stored in the “contouring” section of the IPR Set, the instrument 850.314.012 is used prior to final polishing for contouring, smoothing and bevelling of edges and beads. This instrument is designed to give the dental surfaces a natural, aesthetically pleasing look.

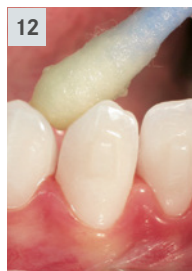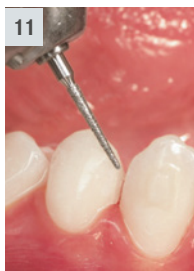

12. After polishing, apply fluoride to the enamel to conclude the treatment.

Recommendations:

- The segmented discs are exclusively designed for use in the oscillating Komet contra-angle (OS30).
- The discs are used at maximum setting of the micro motor ($\text{r}_{\text{max.}} 40.000 \text{ rpm}$).
- The use of an air motor is also possible ($\text{r}_{\text{max.}} 20.000 \text{ rpm}$).
- Start the oscillating discs before they are applied to the tooth.
- Insert the segmented discs from occlusal and guide down slowly.
- Make sure to use sufficient waterspray coolant at all times (at least 50 ml/min.).
- Segmented discs with coating on the face side are identified by the letter "V" (standing for "vorne", the German word for "face side") on their shanks, whereas discs with a coated rear side are marked with the letter H (= "hinten", German for "rear"). The thickness of the disc is also laser marked on the shank.
- Segmented discs coated with medium diamond grit can be identified by a blue ring, whereas discs coated with fine diamond grit bear a red mark.
- To complete the treatment, the interproximal enamel should be polished and fluoride should be applied to the enamel.

Advantages:

- Much faster and more effective than manual enamel reduction with diamond strips.
- No risk of injury to soft tissues.
- The enamel can be reduced by exactly the required amount.
- Patented oscillating discs for unobstructed view and excellent chip removal.
- Clearly arranged, sterilizable instrument tray

Elimination of the interproximal contact

● **WS37EF** ● **WS37**

IPR 0,2 mm

- **OS1FV.000.140** 0,13 mm
- **OS1FH.000.140** 0,13 mm
- **OS15FV.000.140** 0,15 mm
- **OS15FH.000.140** 0,15 mm
- **OS1F.000.140** 0,15 mm

IPR 0,3 mm

- **OS20FV.000.140** 0,20 mm
- **OS20FH.000.140** 0,20 mm
- **OS20F.000.140** 0,20 mm
- **OS25M.000.140** 0,25 mm

IPR 0,4 mm

- **OS1M.000.140** 0,30 mm
- **OS35M.000.140** 0,35 mm

IPR 0,5 mm



● **8392.314.016**

Contouring



850.314.012



ASR-Set 4594



OS30