




# A new level of flexibility. **F6 SkyTaper®**


Endodontics  
Mechanical  
root canal preparation  
with one file

**STERILE | R**



Komet Products  
Made in Germany

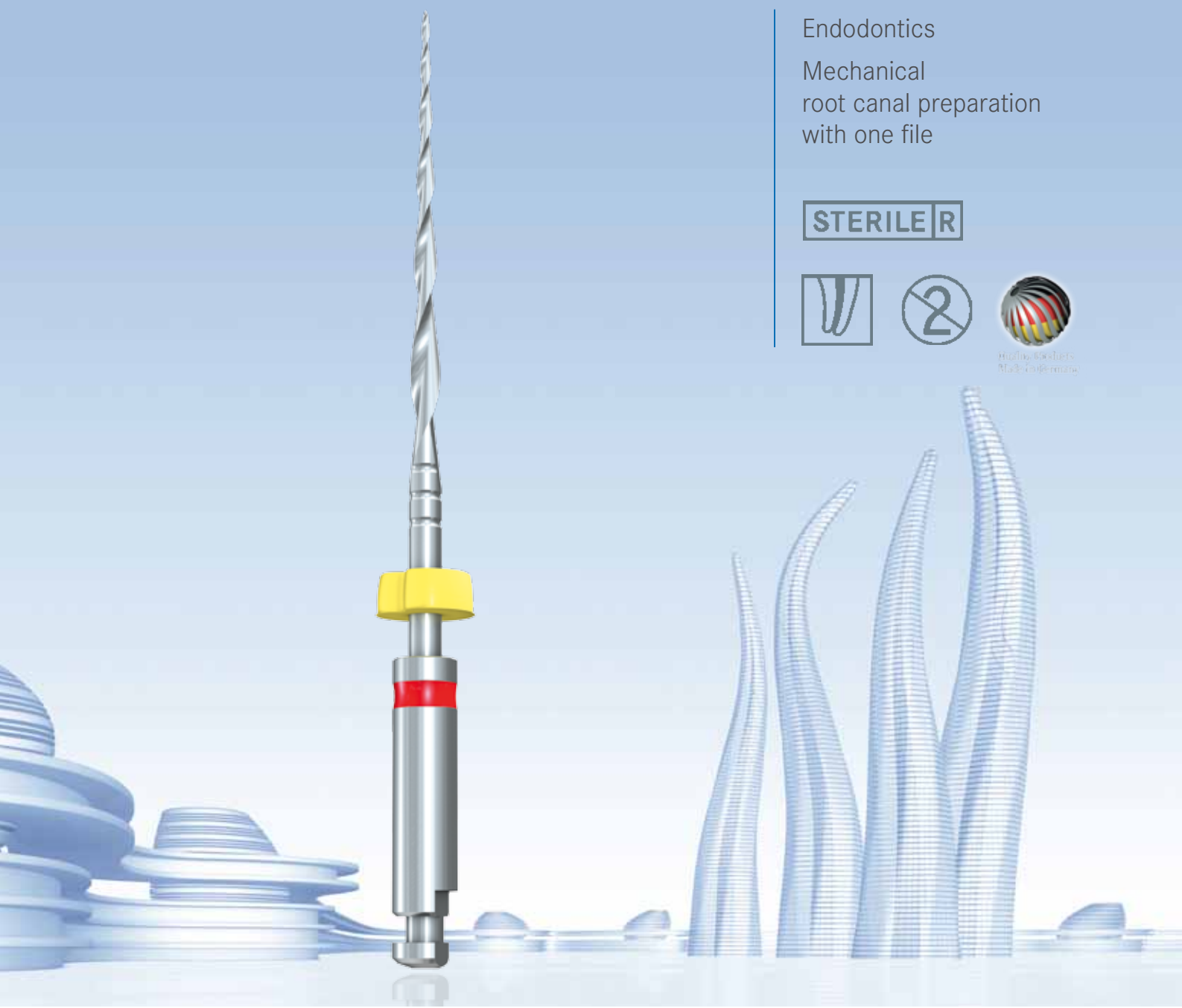

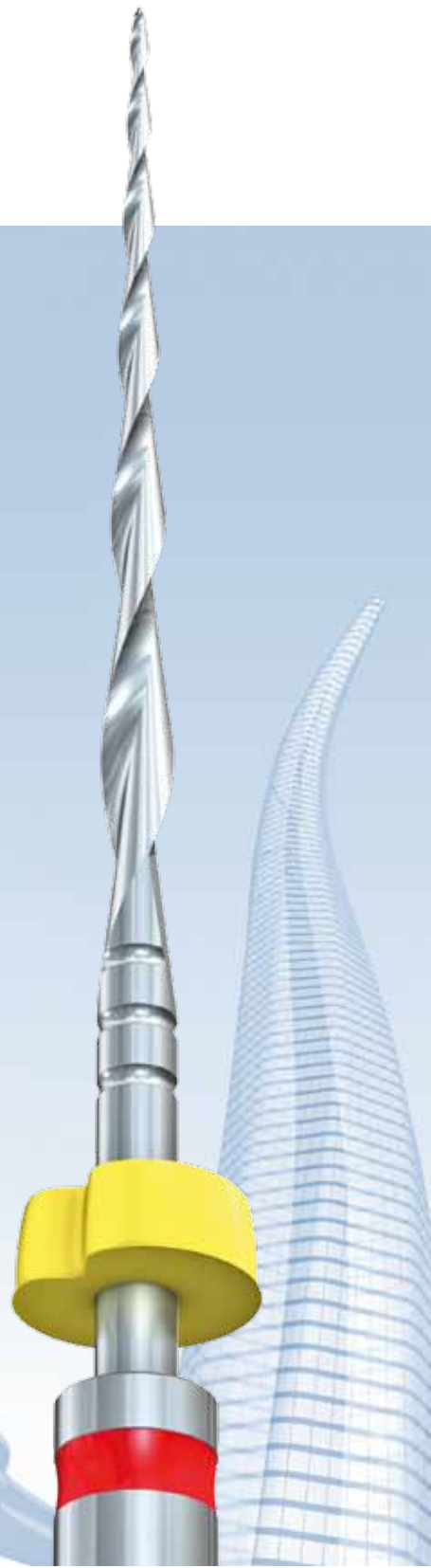

Some of the products and designations mentioned in the text are trademarked, patented or copyrighted. The absence of a special reference or the sign ® should not be interpreted as the absence of legal protection.

This publication is copyrighted. All rights, also with regard to translation, reprint and reproduction (also in the form of extracts) are reserved. No part of this publication may be reproduced or processed using electronic systems in any form or by any means (photocopying, microfilm or other methods) without the written permission of the editor.


Colours and products subject to alterations.  
Printing errors excepted.

As at January 2015



# Larger taper, limitless potential.

- Rotary preparation with one taper .06 file
- All-comprehensive range of files in 5 sizes to suit all root canal anatomies
- Flexibility and efficiency thanks to an intelligent file design
- Sterile packed single-use files

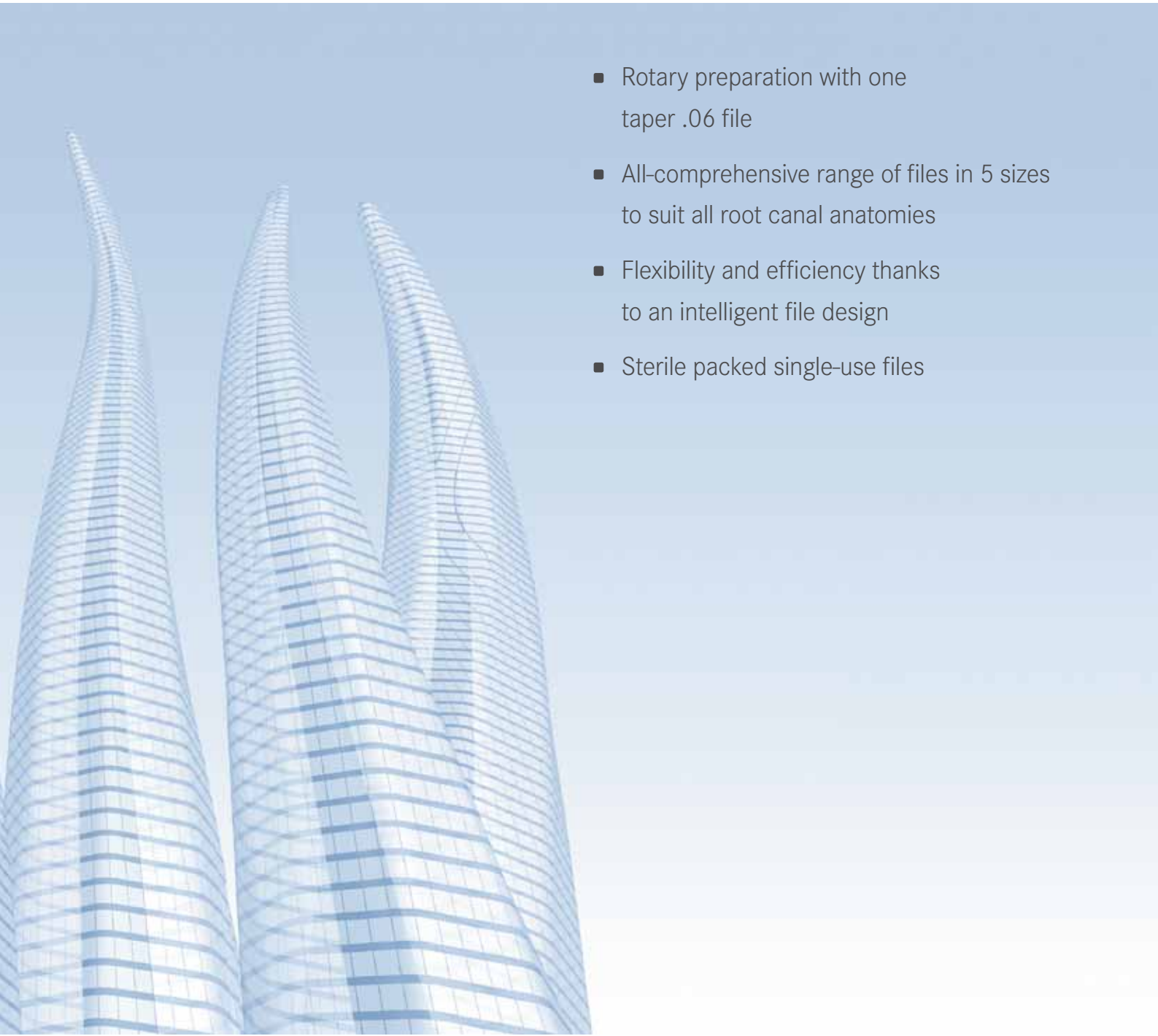



# Flexibility redefined.

## Flexible system and application

- All-comprehensive system: NiTi files in five different sizes and 3 lengths to suit every canal anatomy
- Unrestricted choice of the obturation method thanks to the taper .06
- Rotary use in all torque-limited endodontic contra-angles and motors

## Efficient and flexible file design


- One file for the majority of root canals
- Efficient double-S cross-section for thorough cleaning
- Excellent preservation of the course of the canal

[ 01 ] Generous preparation of the entrance of the root canal, e.g. with the opener OP10L19.

[ 02 ] Creation of a glide path, for example with a K-file in size 015.

The suitable F6 SkyTaper file is chosen according to the size of the previously used (manual) file.  
Rule: Skip one ISO file size

[ 03 ] Preparation of the root canal with the F6 SkyTaper file.

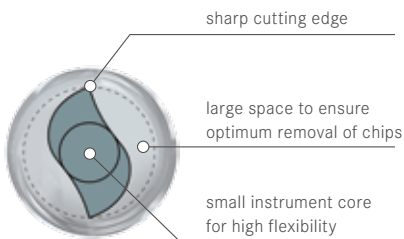

File cross-section



020

025

3

↻<sub>opt.</sub> 300 rpm  
 020 - 030  
 → Torque: 2,2 Ncm  
 035 - 040  
 → Torque: 2,8 Ncm

STERILE R



**Dr. Günther Stöckl**


Specialist in endodontics,  
Rottenburg an der Laaber

*“The F6 SkyTaper system is suitable for both general dentists and specialists. The excellent cutting performance of the instruments shortens the preparation time so that I can spend more time on the chemical disinfection.”*

*“I’m especially impressed with the 6% taper which prepares the canal systems particularly well for the subsequent hot or cold obturation. The F6 SkyTaper system has become an indispensable part of my preparation routine!”*



# F6 SkyTaper. Perfection in every detail.

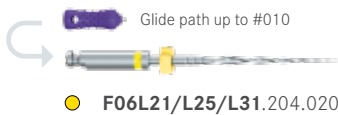

Glide path up to #010  
**F06L21/L25/L31.204.020**



Glide path up to #015  
**F06L21/L25/L31.204.025**



Glide path up to #020  
**F06L21/L25/L31.204.030**




+ Optional  
for wide canals



**F06L21/L25/L31.204.035**



**F06L21/L25/L31.204.040**

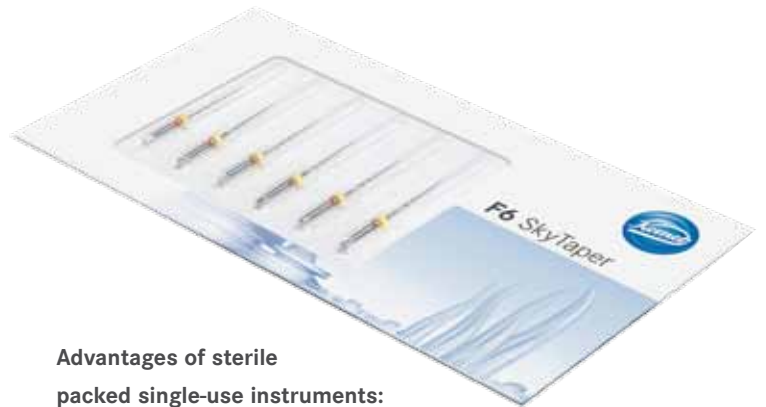

### Advantages of sterile packed single-use instruments:

- Reduced risk of fracture caused by cyclic fatigue
- The risk of cross-contamination is excluded
- No time spent on reprocessing (cleaning, disinfection and sterilization) and recording the frequency of use



**9938 E-Drive**  
Endodontic contra-angle

**Accessories for preparation:**




● **OP10L19.204.030**  
Opener



○ **PG03L21/L25.204.015**  
PathGlider



● **PG03L21/L25.204.020**  
PathGlider

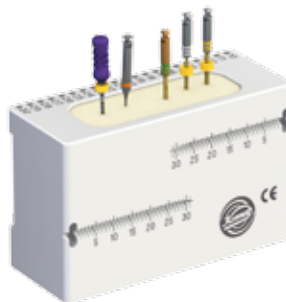

**595** Interim bur block

**Accessories for obturation:**



**9994** F360 Fill Oven



**9996** F360 Fill Obturators



**9995** F360 Fill Verifier



**PPF06** F6 SkyTaper paper points

**GPF06** F6 SkyTaper gutta-percha points



**9978** EasySeal  
Root filling material based on epoxy resin

Komet Dental

Gebr. Brasseler GmbH & Co. KG

Trophagener Weg 25 · 32657 Lemgo

Postfach 160 · 32631 Lemgo · Germany

Verkauf Deutschland:

Telefon +49 (0) 5261 701-700

Telefax +49 (0) 5261 701-289

info@kometdental.de

www.kometdental.de

Export:

Telefon +49 (0) 5261 701-0

Telefax +49 (0) 5261 701-329

export@kometdental.de

www.kometdental.de

Komet Austria Handelsagentur GmbH

Innsbrucker Bundesstraße 75

5020 Salzburg · Austria

Telefon +43 (0) 662 829-434

Telefax +43 (0) 662 829-435

info@kometdental.at

www.kometdental.at

