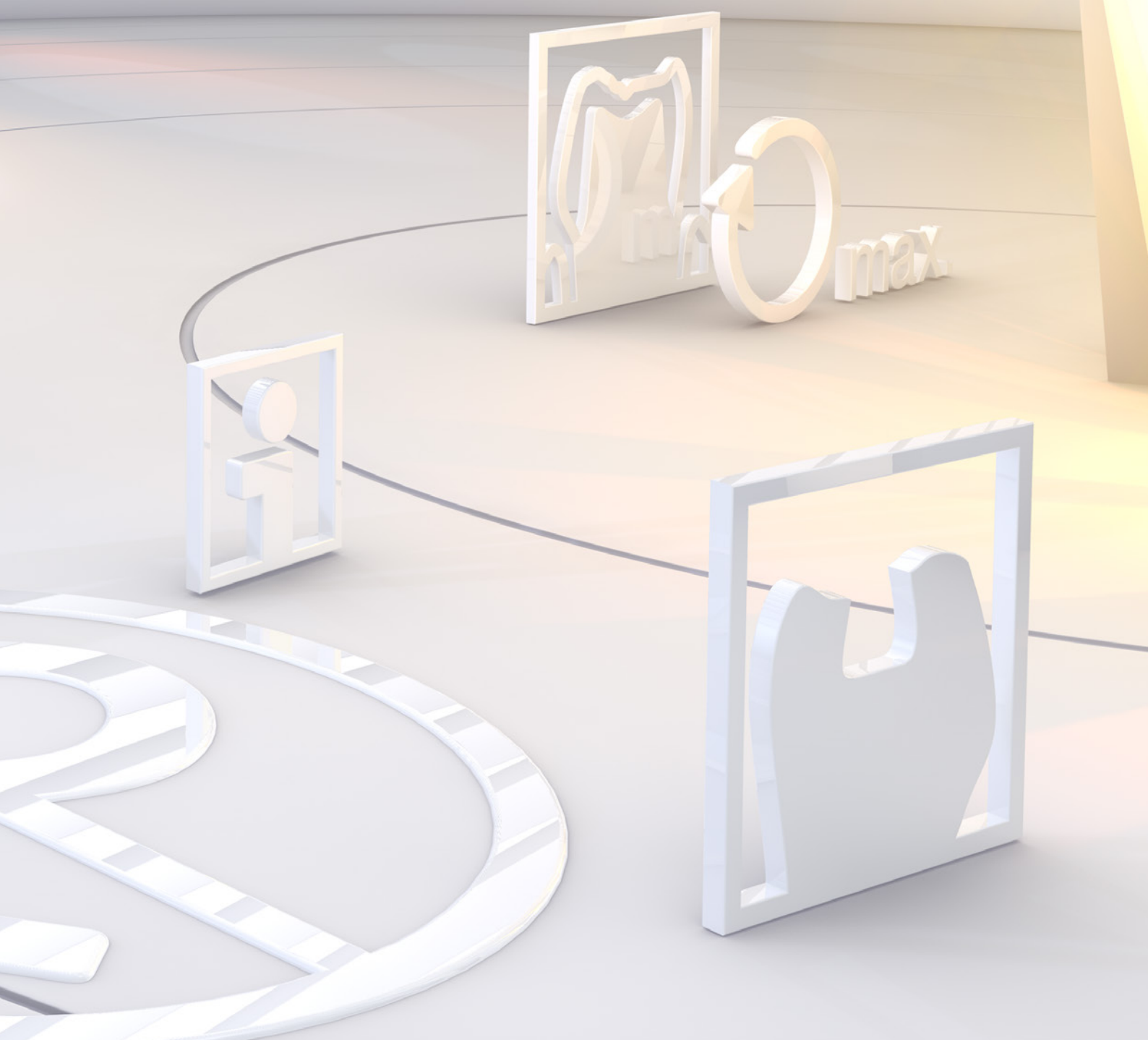
Bruksanvisning og sikkerhedsanbefalinger
Instructions for use and safety recommendations

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Instructions for use and safety recommendations

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Per mars 2023

Some of the products and designations mentioned in the text are trademarked, patented or copyrighted. The absence of a special reference or the sign ® should not be interpreted as the absence of legal protection.

This publication is copyrighted. All rights, also with regard to translation, reprint and reproduction (also in the form of extracts) are reserved. No part of this publication may be reproduced or reprocessed using electronic systems in any form or by any means (photocopying, microfilm or other methods) without the written permission of the editor.

Colours and products are subject to alterations. Printing errors excepted.

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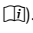
General instructions for use and safety recommendations for the application of dental instruments, pins, posts and bur blocks

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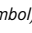
Area of application

These general instructions for use and safety recommendations apply to all products and have to be generally observed. Non-observance of these instructions for use and safety recommendations increases the risk of injury and may impair the proper function of the instruments.

Separate instructions for use are enclosed in the packaging of products that require more detailed information (packaging is provided with the  symbol). These take precedence over the general instructions.

Storage

Store wrapped devices protected from UV radiation and high temperatures in a clean and dry environment. Do not store in the same room as solvents or chemicals.

Medical devices without direct identification marking have to be stored with their original packaging.

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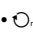
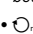
1. Proper use

- Those instruments that are supplied non-sterile have to be prepared prior to first use.
- Make sure that the power systems (hand-pieces and contra-angles) used are clean and perfect from a technical and hygienic aspect.
- Depending on their shank type, insert the instrument into the chuck as deeply as possible. Make sure that they are properly locked.
- The instrument must be rotating at the desired speed before contact is made with the tissue or material.
- Avoid jamming and using the instrument as a lever. Excessive contact pressure has to be avoided.
- For eye protection wear safety glasses. Use appropriate respiratory protection (mouth and nose). In the dental laboratory, use appropriate suction unit.
- Avoid contact with the instruments' working parts as this may increase the risk of injury.
- Make sure to provide sufficient cooling by means of air/water spray. Additional external cooling is required when using instruments with extra-long shanks or oversized working parts.
- Damaged or corroded instruments have to be discarded.

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2. Recommended speeds

- Please make sure to observe the recommendations for use and recommended speeds as indicated in the instructions for use and on the packaging of the products.
-  300 000 rpm means: Suited for micro-motor contra-angles and turbines with stable ball bearings. Do not use in turbines with air bearing.
 -  200 000 rpm means: Suited for micro-motor hand pieces and contra-angles or lab hand pieces up to the speed indicated. Do not use in turbines.
- Not observing the maximum permissible speed leads to an increased safety risk.

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3. Contact pressure

- Excessive contact pressure (> 2N) has to be avoided.
- In cutting instruments, this can lead to damage to the working part and to chipping of the blades as well as an excessive generation of heat.
 - In abrasive instruments, increased contact pressure may lead to stripping of the grit or to clogging of the instruments and increased heat generation.
 - Increased contact pressure may also lead to thermal damage to the pulp or, in case of damaged blades, to rough surfaces.
 - In the extreme cases, instrument breakage may even occur.

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4. Guideline on the number of times rotary and oscillating instruments can be used

The below values are guidelines. The service life of instruments may differ from these values as this depends on the application and/or the material treated.

Stainless steel instruments:	- up to 4 x
Tungsten carbide instruments:	- up to 15 x
Diamond and ceramic instruments:	- up to 25 x
Polishers and ceramic abrasives:	- up to 10 x
Sonic tips/ultrasonic tips:	- up to 25 x
Endodontic instruments: Wide canals:	- max. 8 x
Average canals:	- max. 4 x
Narrow canals:	just use 1 x

The reuse of disposable products is not permitted. These are intended for single use and cannot be reprocessed.

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5. Disposal

The devices have to be disposed of in tight, non-breakable and puncture-proof containers (protection from contamination).

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6. Disinfection, cleaning and sterilisation

Those instruments that are supplied non-sterile have to be prepared prior to first use. Information on reprocessing, as amended, can be downloaded from our website <http://ifr.kometdental.com> or requested from the manufacturer Gebr. Brasseler.

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7. Specific instructions for individual instrument types

- Avoid any contact with H₂O₂ (hydrogen peroxide). Make sure that the specified immersion times in the cleaning and disinfecting agents are not exceeded. The carbide working parts would be attacked (a typical indication is black staining of the instrument) reducing the instrument's service life.
- Tool steel instruments corrode and can therefore not be sterilized with a sterilization method using saturated steam (e. g. autoclave).
- To achieve an optimal surface roughness, subsequent finishing is necessary after using a diamond grinding instrument with coarse or very coarse grit.
- Use a disc guard for diamond discs when working intraorally.
- When using diamond separating strips and diamond strips please avoid contact with the gingiva as there is a risk of injury.
- Due to the ergonomically flat design of reciprocating files, please avoid jamming, bending or using the file as a lever during use. Please make sure that the files are properly fixed in the chuck of the reciprocating contra-angle to the required depth.
- To avoid overstraining of the instrument, root canal instruments made of nickel-titanium have to be used in a torque limited motor.
- Stainless steel root canal reamers (type "Gates", "Müller", "P") are only intended for preparation of the coronal portion of the canal.
- For reprocessing anodized aluminium bur blocks, cleaning and disinfecting agents suitable for aluminium must be used. Other agents would destroy the anodized layer of the bur block. Aluminium blocks are generally not suited for preparation in the thermo disinfecter. Prior to sterilization, rinse bur block under running water and dry thoroughly (e.g. by air blasting). Frequent reprocessing may lead to colour deviations.
- Wipe off any stains on instrument blocks immediately after disinfection.
- Make sure to replace worn or damaged silicone plugs on instrument blocks.
- Stainless steel instrument block 9952 is intended for the storage and sterilisation of sonic tips and accessories.
- Root canal posts made of fibre reinforced composite or PMMA cannot be sterilized. Root posts are intended for single use only.
- Clean and sharpen DSB abrasives every now and then with the cleaning stone.
- Always use polishers and brushes with low pressure to minimize heat generation. Always polish in circular motion. To achieve brilliant high shine, use the polishers in the indicated sequence when using multiphase polishing systems.
- Polishers and Arkansas abrasives have to be prepared with suitable, alcohol-free agents (e.g. DC EVO).
- Discard any ceramic instruments that were subjected to sudden stress (for example by falling off the edge of the table). Do not reuse these as they are particularly prone to fracture.
- Surgical instruments with FG shank are only authorised for use in a micro motor (red contra-angle). Do not use in a dental turbine.

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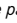
8. Potential effects of using worn instruments

- Discard any instruments with damaged or worn working parts as the use of damaged or worn instruments would have a negative effect on the work result.
- Damaged and deformed cutting blades can cause the instrument to vibrate and lead to poor preparation margins and rough surfaces.
- Void spots on the surface of diamond instruments are a sign of missing diamond particles and reduce the instrument's abrasive efficiency. An inferior diamond coating quality will result in excessive heat generation. Excessive contact pressure or temperatures may cause damage to the tissue.
- Please avoid excessive contact pressure during use as this may result in damage to the working part (Nicks on the blades, premature blunting and excessive heat generation).
- In abrasive instruments, excessive contact pressure can lead to stripping of the grit or clogging of the instruments and increased heat generation.
- To avoid undesirable heat generation during preparation, make sure to provide sufficient cooling by means of air/water spray (at least 50 ml/min).
- Additional external cooling is required when using instruments with a total length of more than 22 mm or a head diameter exceeding 2,5 mm.
- Not observing the maximum permissible speed will result in an increased risk of injury.
- The use of reusable instruments that have not been reprocessed properly increases the risk of infection.
- The reuse of disposable instruments (marked  on the packaging) is not permitted (e.g. polishers with lamellae and dental brushes). The reuse of these products poses a risk of infection and/or the safety of the products can no longer be guaranteed (e.g. due to the risk of fracture with root canal instruments).

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9. Safety and possible side effects

The above mentioned recommendations with respect to cooling, contact pressure, disinfection, cleaning and sterilization are to be strictly observed. The instruments should only be used for the intended application, symbolized by the respective icon. Non-observance of these safety recommendations may lead to damage of the power system and/or injury, such as thermal necrosis, undesired preparation of tissue, damage to tissue or nerves, violation of the biological width, or infections. During preparation, some instruments may generate metallic abrasion which may lead to the presence of artifacts in MRI diagnostics.

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10. Liability

The user is solely responsible for checking the devices prior to use to ensure that they are suited for the intended purpose. The correct application of the devices is the responsibility of the user. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability towards the user for all resulting damages, particularly if these are due to the non-observance of the recommendations for use or other misuse by the user. Store out of children's reach. For dental use only.

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11. Contains hazardous substances

Devices marked with a CAS number (CAS: 7440-48-4) contain more than 0.1 % weight percent of cobalt. As a CMR substance of class IB, cobalt is classed as possibly carcinogenic, mutagenic and/or toxic to reproduction. Evidence has shown that the quantities of cobalt released by medical devices are so low that they do not pose a risk and no precautions have to be taken, provided that the medical device is used correctly and according to its intended purpose.

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12. Procedure in case of serious adverse events

In case of serious adverse events, please proceed in line with the legal regulations in your country.

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Kun til engangsbruk*
For single use only*

Inneholder farlige stoffer
Contains hazardous substances

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¹ After the expiration date, the use of sterile packed instru-
ments carries a risk of infection, and the proper function
of the product can therefore not be guaranteed.

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² Do not use in case of damaged packaging as the
sterility of the product is no longer guaranteed.
Make sure to check that reusable instruments are in
perfect condition and reprocess prior to use.

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* The reuse of these products carries a risk of
infection. A safe, risk-free use can therefore not be
guaranteed.

Antall blader – wolframkarbidinstrumenter for finishing · Number of blades for carbide finishers

○	ultrafin · ultra-fine	30 blader · blades
●	fin · fine	16/20 blader · blades
●	normal · normal	8/12 blader · blades

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* With some instruments the grit size may
deviate from the specified value, depending
on their shape and size.

Komet Dental

Gebr. Brasseler GmbH & Co. KG

Trophagener Weg 25 · 32657 Lemgo

Postfach 160 · 32631 Lemgo · Germany

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Telefon 0800 7701700

Telefax 0800 7701800

info@kometdental.de

www.kometdental.de

Export:

Telefon +49 (0) 5261 701-0

Telefax +49 (0) 5261 701-329

export@kometdental.de

www.kometdental.com

Komet Austria Handelsagentur GmbH

Hellbrunner Straße 15

5020 Salzburg · Austria

Telefon +43 (0) 662 829-434

Telefax +43 (0) 662 829-435

info@kometdental.at

www.kometdental.at